


Drug screening has never been
so clean, easy and cost effective.



Available tests include:

- AMP, M-AMP, BAR, BZO, COC, MDMA, MTD, OPI, OXY, PCP, TCA and THC.

REDICUP COMPONENTS

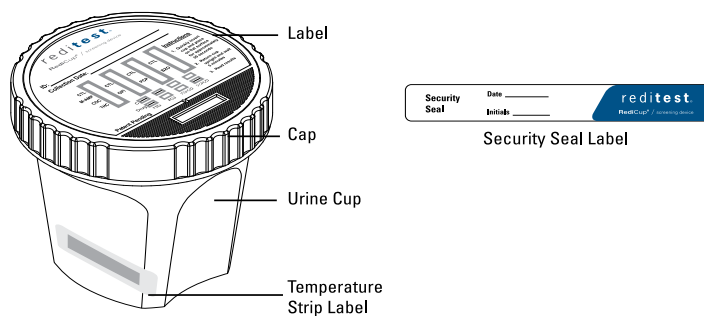

TESTING PROCEDURE

- 1** Collect specimen.
- 2** Secure cap. **NOTE:** Screw cap tightly to ensure adequate seal.
- 3** Quickly invert cup and place on a flat surface for approximately 30 seconds.
- 4** Stand upright on a flat surface. If no sample wicking is seen at 1 minute, repeat steps 3 and 4.
- 5** Read results at 5 minutes. Do not interpret results after 60 minutes as false results may occur. Note: Any line at all in the test zone is to be interpreted as negative.

RESULT KEY

C				
Drug				
	NEG	POS	INVALID	INVALID

*NOTE: ANY POSITIVE RESULT OBTAINED WITH THIS URINE SCREENING TEST IS PRESUMPTIVE AND SHOULD BE CONFIRMED BY AN ALTERNATE METHOD SUCH AS GC/MS. CONTACT OUR LAB FOR ADDITIONAL GC/MS CONFIRMATION SERVICES.

For professional in vitro diagnostic use only.

*Amphetamine, Barbiturates, Benzodiazepines, Cocaine Metabolites, Marijuana (THC), MDMA (Ecstasy), Methadone, Opiates (300 or 2,000 ng/mL cut-off), Oxycodone, Phencyclidine, and Tricyclic Antidepressants (TCA).

Call today: **877-444-0049**